

Hip Belt Extension: Universal

The ISO Preferred Lumbar Belt Extension: Universal provides an extension to the hip belt for waist sizes over 56 inches.



ISO Preferred Hip Belt Extension: Universal (HI204) Applies to two different types of Hip Braces, the ISO Preferred Post OP Hip Brace & The Clam Shell models.

ISO- HI203 Post Op Hip Brace W/ Flexion/Extension/Abduction

1. Detach all hook and loop fastening at lumbar support and lay on flat surface.
2. Attach extension with tag face up to the side of the brace that also has a tag, make sure it is securely fastened. Apply brace as instructed on product instructions and attach other side of extension to desired position.



CLAM SHELL MODELS: ISO-HI205 ISO-HI206

BEFORE APPLICATION: Detach both belt wraps from the back of the Lumbar Belt.

1. **APPLY THE EXTENSION:** Open up the clam shell fastener and apply extension facing down.
2. Take the piece of the original belt and attach to belt at desired size securely. **MAKE SURE SIDE WITH THE HOOK AND LOOP FASTENING IS ON THE RIGHT SIDE OF THE BODY.** Apply to the other side of the brace making sure both sides are even. Apply brace as instructed on product instructions.



INTRODUCTION

Your **ISO Preferred Hip Belt Extension: Universal (HI204)** is designed to provide your body with the support and stability that it needs to successfully manage or eliminate your mobility or pain concerns. It is critical that you follow your orthopedic professional's directions closely to ensure that the apparatus is worn properly and in the manner that will be most beneficial in addressing your issues. For correct application, please read the enclosed instructions and watch the on-line instructional video at www.ISOpreferred.com so that this device can provide you with superior long-term usability and care.

USAGE RECOMMENDATIONS

It is vital that you read the included instructions and when applicable watch the instructional video for the **ISO Preferred Hip Belt Extension: Universal (HI204)** that is included here within. It is intended for the use of qualified professionals only. It is not recommended for patients to fit or apply or adjust their new brace without an orthopedic professional's expertise, unless otherwise instructed. Further questions in regards to adjustments or changes, frequency of use, and/or duration of use should be presented to your orthopedic professional, as they are familiar with your needs and conditions, for which this device was prescribed.

CARE AND CLEANING INSTRUCTIONS

Please do not place your brace in the washing machine or dryer at any time. When preparing to clean your **ISO Preferred Hip Belt Extension: Universal (HI204)** you will want to separate the parts where applicable and care for them individually.

- Do not dry clean, iron, bleach, or use an abrasive cleaning instrument on any part of the brace. Failure to follow these instructions may damage your brace and void the manufacturer's warranty and it may no longer work properly.
- Hand wash each section in cold water with a mild detergent. Lay flat to air dry. Allow all parts to air dry prior to reassembling the device.

CAUTION

The **ISO Preferred Hip Belt Extension: Universal (HI204)** is for single patient use only. While every effort has been made to obtain maximum strength, function and durability, there is no guarantee that injury will be prevented through the use of this device, especially if the instructions are not followed correctly.

DO NOT OVERTIGHTEN.