

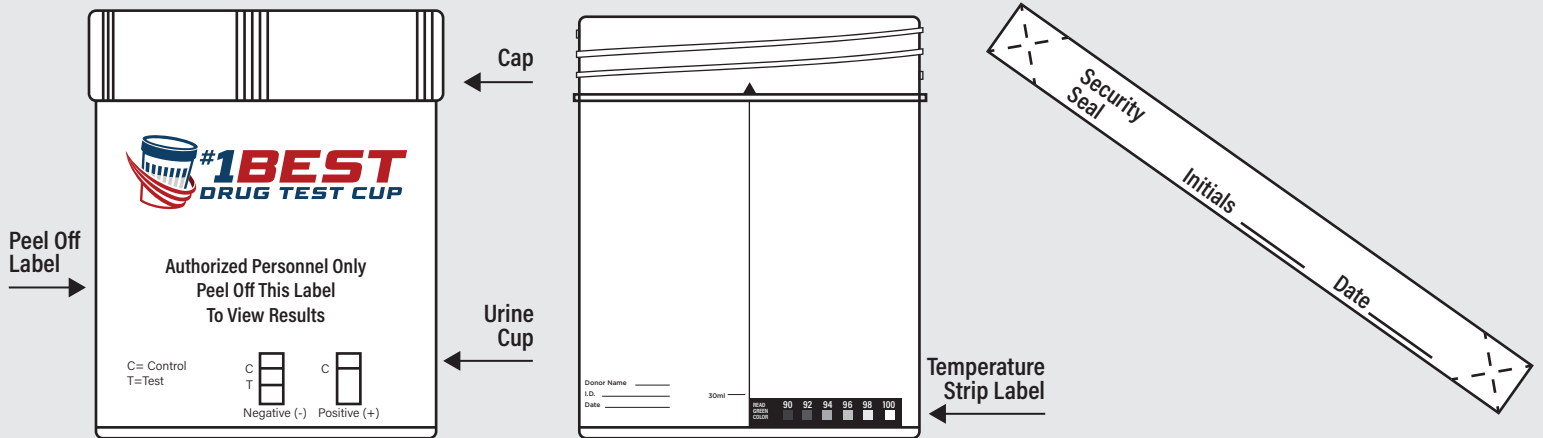


# #1 Best Drug Test Cup Procedure




## Components and Materials

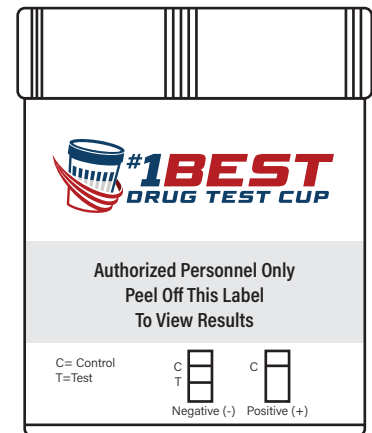

1. Remove the cap.
2. Donor provides specimen. Pour into cup.
3. Place the cap flat on top of the cup.

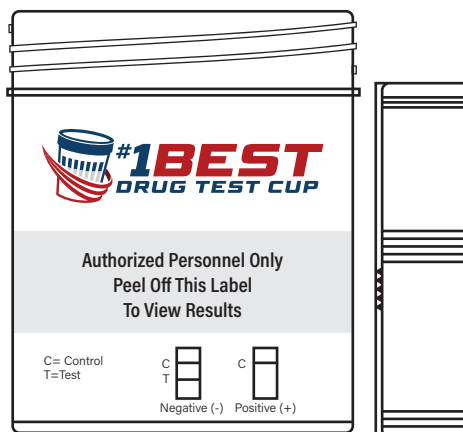


4. Twist the cap clockwise until you hear a 'click' indicating the system is locked.
5. Check to make sure the cap is completely and tightly closed, and the cup is locked. **IMPORTANT:** The cup **MUST** be locked prior to shipping to laboratory for confirmation testing.

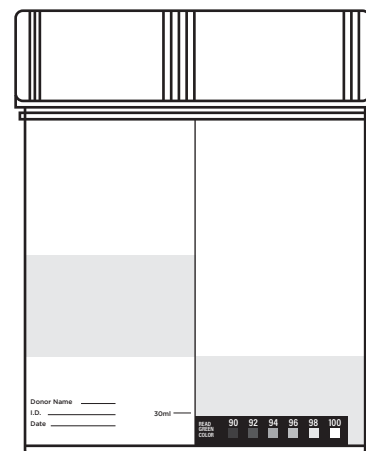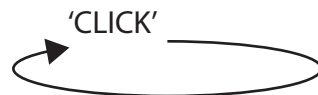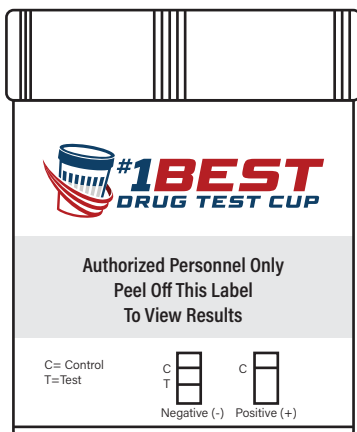


**ENSURE TIGHT SEAL**

# #1 Best Drug Test Cup Procedure


**6.** Place security seal over lid after donor initials & dates. Do not place seal over results window.

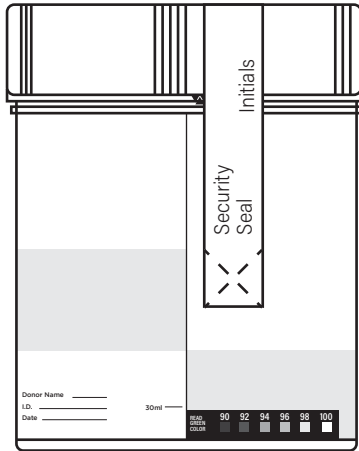


**7.** Read urine temperature between 2-4 minutes after collecting. Normal urine temperature should fall in the ranges from 90-100°F (32-38°C).

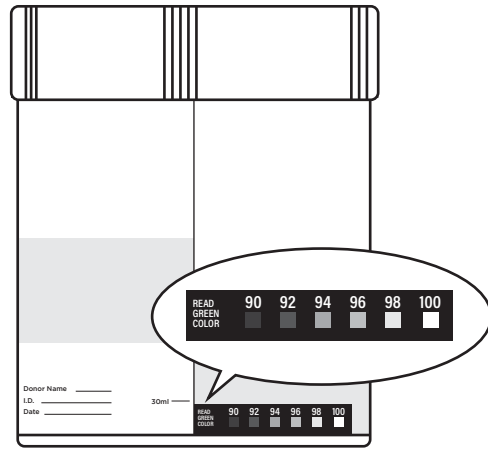

**8.** Wait 5 minutes after introducing sample to cup.

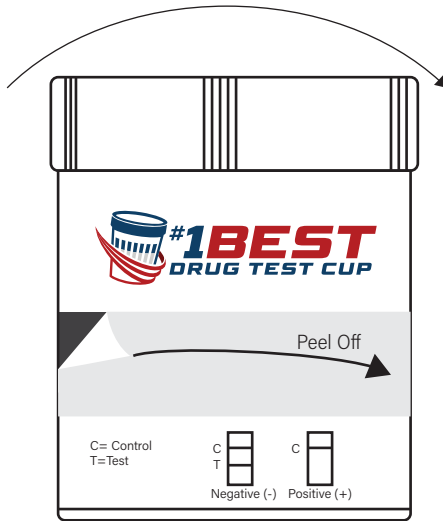


**10.** Peel Label and Read the drug test results at 5 minutes.

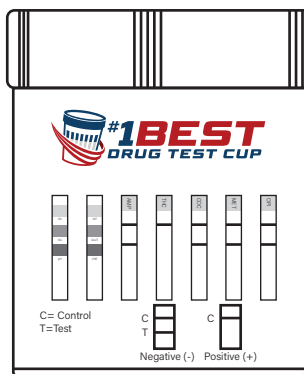

← Drug Test Names  
 ← C Line in "C" Region  
 ← T Line in "T" Region



## Read Results at 5 Minutes

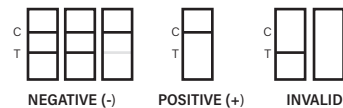

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)